HANDELINGEN EERSTE HULP

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FIRST AID MEASURES

With injuries to arms or legs, remove any jewellery as quickly as possible to prevent constriction in case of swelling.

Important rules for First Aid:	Calling 112
1. Beware of danger.	When you call for help, mention:
2. Assess the situation and then the casualty.	- Which service you require
3. Reassure the casualty and provide protection.	- What has happened
4. Get professional help.	- Where it happened
5. Help the casualty at the place where he lies or sits.	- Whether there are several casualties
	- What is wrong with the casualties

Vomiting when unconscious. Someone is choking.	Turn an unconscious casualty who is lying on his back onto his side. Give 5 slaps between the shoulder blades. Slap with a flat hand. Support the chest with your other hand.
A wound bleeds a lot.	Stop the bleeding. Apply pressure to the wound. If possible, put sterile dressing or a clean towel on the wound first.
For burns.	Cool the burns. Cool for at least 10 minutes. Let lukewarm tap water to flow gently over the burnt area.
For an abrasion (gaze).	Rinse the wound with clean water. Only cover the wound if necessary, e.g. because of friction from clothing.
A body part hurts and cannot be used. (dislocation/ bone fracture)	Prevent movement. Use an adhesive bandage or a sterile compress, preferably bon-stick. Let the casualty support his own arm. Support a leg along its entire length, including the foot.
A body part hurts but can be moved. (bruise/sprain)	Cool for 10 to 15 minutes. Preferably with a bag of water and ice, cold running tap water, a wet towel or a coldpack. Give support and elevate the injured body part, if possible.
You have used this first aid kit.	Replace materials quickly Contact your supplier.

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Keep this manual inside the first aid box

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First Aid Orange Cross

The first aid kit BHV contains all the necessities for a single work unit.

The content of the kit is in line with the skills of a professional caregiver. Also people without specific first aid knowledge can use the kit to take care of minor injuries.

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